




Directions for uploading custom icons to the T-Scope library:

1. Click the "Download D2 Content" button.
2. By default, the zip file will save to your downloads folder (you can also download it anywhere on your computer).
3. After the zip file named "T-Scope ICONS.zip" has downloaded, here are the steps to unzip the file:
 - On a Windows PC
 - Find the "T-Scope ICONS .zip" file
 - Right -click on it and select "Extract All"
 - Follow the wizard to choose a destination on your computer
 - Then click "Extract" to get the files unzipped
 - On a MacOS
 - Open Finder and find the zip file you want to open
 - Double-click the "T-Scope ICONS .zip" file and the built-in archive utility will automatically extract the file contents into a new folder in the same location; or right-click (or Control-click) the file and select "Open With" and then "Archive Utility: if double-clicking doesn't work"
4. Click on the folder named "T-Scope ICONS" and select the desired icons and copy them to a USB flash drive.
5. Insert flash drive into a USB port on the Netbook computer.
6. Log in to D2 software.
7. Click *Experiment with Light Board*.
8. Under the T-Scope option, click *Change*.
9. Click *Advanced*.
10. Click *Edit Library*.
11. Under the drop down menu titled *Edit What?* select *Icons*.
12. Click *New List*.
13. Choose an appropriate name for your list (i.e. ABC, or 123).
14. Click "Modify List"
15. Click "Import"
16. Locate desired content on the flash drive.
17. Click on each individual character to upload.
18. Click *OK* two times to return to the Advanced T-Scope Menu.

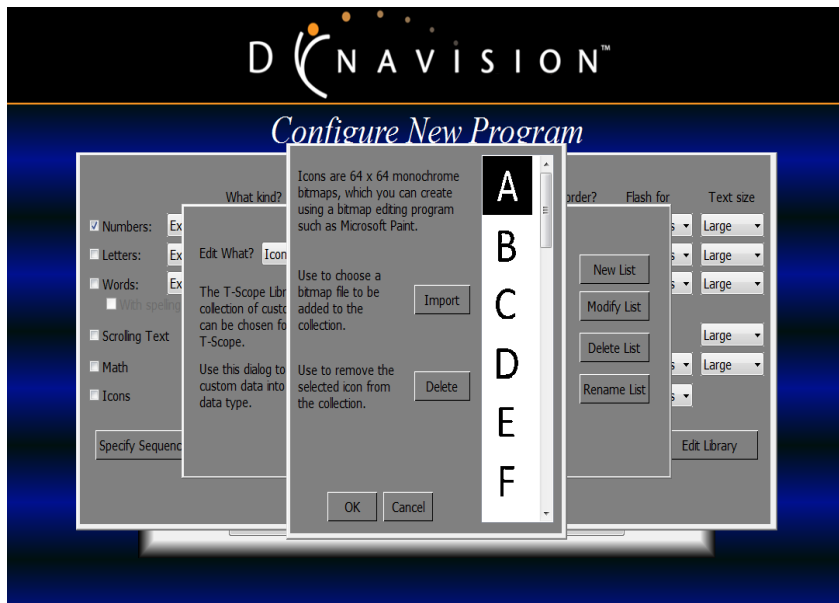

To choose what will be shown on the T-Scope:

1. Under the Advanced T-Scope menu, check the box titled *Icons*.
2. Click the drop down menu titled *From Where?* (currently set to *Default Icons*)
3. Select the name of your list (i.e. ABC or 123).
4. Click *OK*.

