




Directions for uploading custom icons to the T-Scope library:

1. Download desired icons to a flash drive.
2. Insert flash drive into a USB port on the Netbook computer.
3. Log in to D2 software.
4. Click *Experiment with Light Board*.
5. Under the T-Scope option, click *Change*.
6. Click *Advanced*.
7. Click *Edit Library*.
8. Under the drop down menu titled *Edit What?* select *Icons*.
9. Click *New List*.
10. Choose an appropriate name for your list (i.e. ABC, or 123).
11. Click "Modify List"
12. Click "Import"
13. Locate desired content on the flash drive.
14. Click on each individual character to upload.
15. Click *OK* two times to return to the Advanced T-Scope Menu.

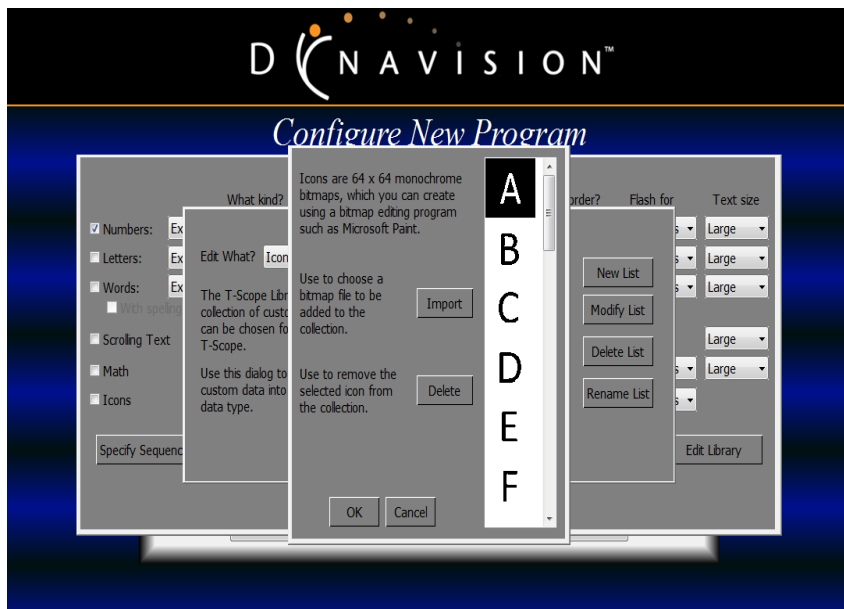

To choose what will be shown on the T-Scope:

1. Under the Advanced T-Scope menu, check the box titled *Icons*.
2. Click the drop down menu titled *From Where?* (currently set to *Default Icons*)
3. Select the name of your list (i.e. ABC or 123).
4. Click *OK*.

